

Important information on how to change a Freka® Click Adapter, ENFit

Changing of the Freka® Click Adapter



1 Freka Intestinal Feeding Tube, ENFit




2 To remove the Click Adapter squeeze the contoured surfaces of the Click Adapter firmly between two fingers



2a Pull out the intestinal tube carefully for approx. 3 - 4 cm



2b To prevent the intestinal tube from slipping, close the tube clamp for fixation.

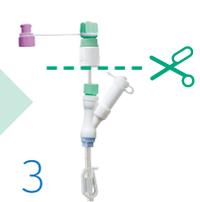

3 Cut off the Click Adapter

Fixation of the intestinal tube to the gastric tube




4 Push the new Click Adapter (green/white) with its white end first over the intestinal tube.




 Metal pin of the Freka Connector, ENFit (green) and the **external end** of the intestinal tube must be **free of grease, clean and dry**. MCT Oil on the metal pin increases the risk of detaching of the intestinal tube.



5 Push the metal pin of the Freka Connector, ENFit **up to its base** into the intestinal tube.

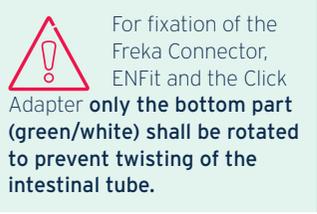


 For fixation of the Freka Connector, ENFit and the Click Adapter **only the bottom part (green/white) shall be rotated to prevent twisting of the intestinal tube.**

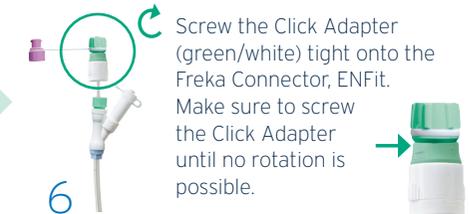

6 Screw the Click Adapter (green/white) tight onto the Freka Connector, ENFit. Make sure to screw the Click Adapter until no rotation is possible.




7 Open the tube clamp of the gastric tube and push the connected Click Adapter over the (green) intestinal access of the Freka Y-Connector, ENFit marked with an "i". The fixation must be clearly audible with a **"Click"**! Close the intestinal tube with the cap of the Freka Connector, ENFit.

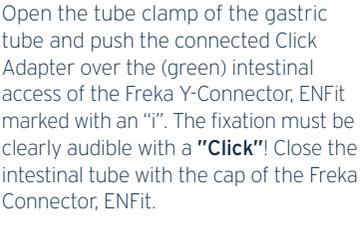

8



 A re-opened Click Adapter, cannot be used again, once the connection was detached. A proper connection cannot be generated again. A new Click Adapter must be used.

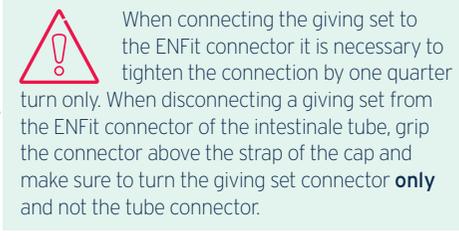

 When connecting the giving set to the ENFit connector it is necessary to tighten the connection by one quarter turn only. When disconnecting a giving set from the ENFit connector of the intestinale tube, grip the connector above the strap of the cap and make sure to turn the giving set connector **only** and not the tube connector.



7395231 (08/18/AC)